ZN/EN-A03

Curing Box FC 3

Operation Manual

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Please tear off the protective film on the inside of the shield door.
使用前请撕去遮光门内侧的保护膜

Notice

Safety notice: Please read all the safety warnings and notice below.

Work Environment Safety

- ◆ Ensure curing box worktable clean and tidy.
- ◆ Do not operate curing box in the presence of flammable liquids, gases or dust.
- ◆ Keep curing box out of children and untrained people's reach.
- ◆ The equipment should be placed in a dry and ventilated environment. The front and rear sides of the equipment must keep distance of at least 20cm, and the space above must keep distance of at least 35 cm.

Electrical Safety

- ◆ Always use the curing box with a properly grounded outlet. Do not refit curing box plug.
- ◆ Do not use curing box in damp or wet locations. Do not expose curing box to burning sun.
In case of device damage.
- ◆ Please use the power supply provided by FlashForge.
- ◆ Avoid using the device during an thunderstorm.
- ◆ In case of uncertain accident, please unplug the device if you do not use it for long.

Personal Safety

- ◆ Do not touch resin directly by hand.
- ◆ Wear gloves and mask when operating the device.
- ◆ Do not look directly at the light source of the curing box.
- ◆ Do not operate the device while you are tired or under the influence of drugs, alcohol or medication.

Caution

- ◆ Operate the device in a well-ventilated environment.
- ◆ Do not leave the device unattended for long.
- ◆ Do not make any modifications to the device.
- ◆ Never use the device for illegal activities.
- ◆ Never use the device to make any food storage vessels.
- ◆ Never use the device to make any electrical appliance.
- ◆ Never put the model into your mouth.

Environment Requirements

- ◆ Temperature: RT 15-30°C; Moisture: 20%-70%.

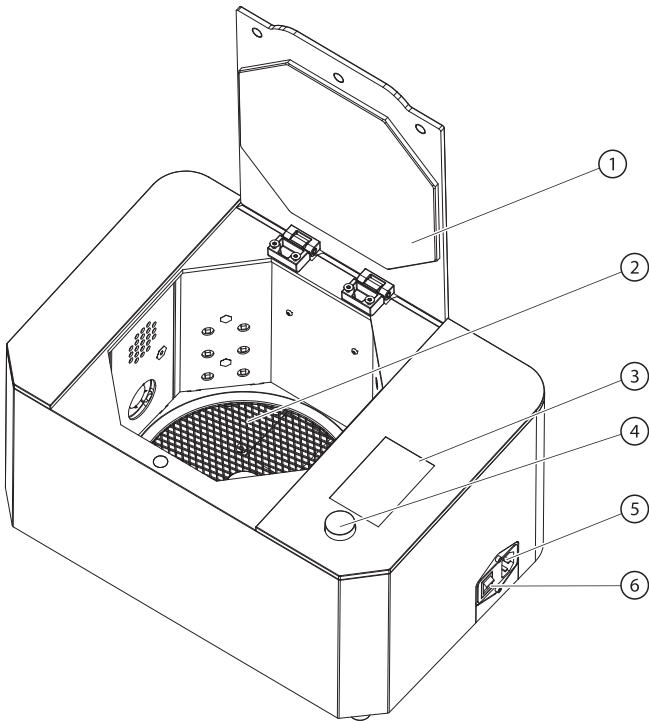
Device storage

- ◆ The equipment should be placed in a dry and ventilated environment. The front and rear sides of the equipment must keep distance of at least 20cm, and the space above must keep distance of at least 35 cm.

Legal notice

- ◆ All the information in this document is subject to any amendment or change without the official authorization from FlashForge.
- ◆ FLASHFORGE CORPORATION MAKES NO WARRANTY OF ANY KIND WITH REGARD TO THIS DOCUMENT, INCLUDING, BUT NOT LIMITED TO, THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE.
- ◆ FCC Notice
This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.
- ◆ FlashForge shall not be liable for errors contained herein for incidental consequential damages in connection with furnishing, performance or use of this material.
- ◆ This document contains proprietary information protected by copyright.
Copyright © 2019 FlashForge Corp. All Rights Reserved.

About Curing Box FC3

1. Shield Door 2. Turnplate 3. Display Screen 4. Manual Knob
5. Power cable input 6. Power Switch

Name	Curing Box FC3
Wavelength	405nm
Curing Volume	φ203*139mm
Dimensions	399*266*225mm
Net Weight	9.7kg
Power requirement	Input: 100V-240V; 4.5A-2A
Rotational Speed	2.5r/min
Power	326W

Description

Operation

After setting the temperature and time parameters, close the door. Turn the knob to "Start" and press the knob. Then the curing box process starts.

Interface Description

Basic Setting

Temp : 35	
Set Temp : 40	
Time : 300	
Temp Set	
Time Set	
Mode	N

Temp: The temperature is the current temperature in the chamber of the Curing Box.

Set Temp: The set-temperature is the previous setting parameter. Automatically remembers the last temperature setting without shutdown.

Time: The last set time.

Temp set: Parameter is 35-60°C.

Time set: Parameter is 0-300 min.

Mode: Curing mode setting. Show the mode selected currently, N is standard mode, and S is strong mode

Normal: Standard mode, most kinds of prints is recommended.

Strong: Strong mode provides higher light strength.

Curing Interface

Temp: 40	N
Curing...	
0:00:55	
Pause	
Cancel	

Temp: The temperature is the current temperature in the chamber of the Curing Box. N means the current curing mode is standard mode.

Curing: Curing is in process.

Count down: Show how much time is left.

Pause: Pause the current curing process. Press the knob, show "Continue".

Cancel: Stop current curing process, back to setting interface.

Finish Interface

Temp: 40	N
Done!	

Done: Curing process is finished with "Beeping".

Warning

Door is open!	
Temp: 40	N
Curing...	
0:00:55	
Pause	
Cancel	

Remind the shield door is not closed. Only shown in the case of pressing knob without closing the shield door.

Attention

During the curing process, if the shield door is opened, it will automatically pause the curing process: heating will stop, the ultraviolet light will be off, and the turn plate will stop rotating. Close the shield door will automatically continue to curing.

Supports and Service

Flashforge team is on standby and ready to help you overcome any challenges you may have with your Curing Box FC3. If the issues or questions are not covered in this User Guide, you can seek for solutions on our official website or contact us via telephone.

There are solutions and instructions to common issues that can be found in our knowledge base. Have a look first as most basic questions are answered there. <http://www.flashforge.com>

The FlashForge support team can be reached by e-mail or phone between the hours of 8:00 a.m. to 5:00 p.m. PST Monday through Saturday. In case you contact us during off-duty time, your inquiry will be answered the following business day.

Tel: 86-0579-82273989

QQ: 2850862986 2850863000 2853382161

Email: support@ff3dp.com

ADD: No. 518, Xianyuan Road, Jinhua, Zhejiang

When contacting support, please have your serial number ready. The serial number is a bar code on the back of your Curing Box FC3.

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