


Wiring instructions for ASV (AutoStart Valves)

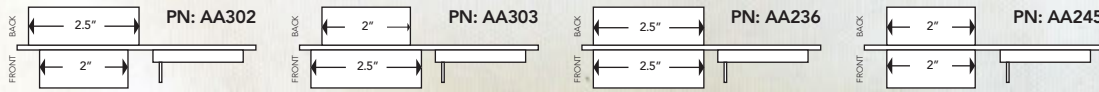

ASV Cable

Note:

- 2" dia is used for Central Vac tubing
- 2.5" dia is used with PVC pipe

Included in the Kit:

- ASV Valve(s)
- DB9 Connector Cable (AE427)

Installation:

1. Run wire from ASV to unit
2. Connect AE427 db9 Connector to the appropriate side socket on the unit
3. Connect **BLACK** wire from ASV to **BLACK** wire from AE427
4. Connect **RED** wire from ASV to **RED** wire from AE427 (you do not require use of any other wire on the AE427 cable)
5. If you have more than 1 ASV, connect all **BLACK wires together** and all **RED wires together**

Operation:

- Dust collector **STARTS** automatically if **AT LEAST ONE ASV** is **OPEN**
- Dust Collector **STOPS** automatically if **ALL ASVs** are **CLOSED**

Switch Designation:

- **BLACK** wire to **COMMON**
- **RED** wire to **NORMALLY CLOSED**

