



Assembly of washing curing unit:


1. Unpack carefully the washing unit.
2. Main basket is attached to the legs.



3. Accessories are illustrated on the next figure.
4. Screw the 3-jets assembly in the threaded hole located on the top of the unit.




5. Install protective walls under the jet assembly.
6. The support for the sprayer is installed directly onto the protective wall or on the wall, at your choice. Connect the sprayer to the water output located on the top of the jet assembly.

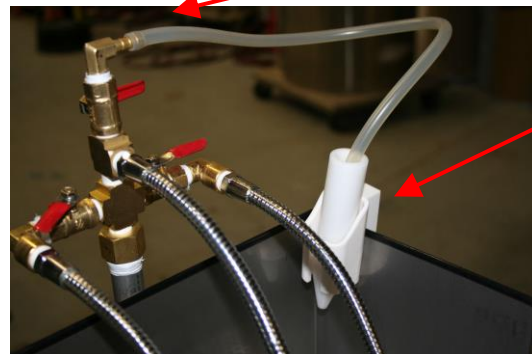



7. Fix the orange waste tubing to the exit valve inside the unit.
8. Unscrew the four screws at the back of curing unit and remount it to form the protective back.

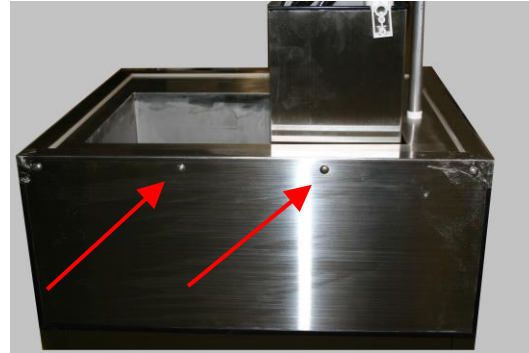

9. The washing unit works on the 220V. Electrical wires are located under the unit on the left. For safety reasons, it is highly recommended to ask a certified electrician to perform the electrical connection.



Operation :

1. Put the main fuse, located inside the door of the unit, to « ON »
2. The yellow switch activates the main power.
3. The green switch turns on the pump..
4. The red switch turns on heating elements.
5. Temperature is set with the large white knob.
6. Timer can be adjusted from 0 to 180 minutes or the unit can be maintained in operation by selecting the « ON » position on the timer knob.