# PHOTO-TRAY / PHOTO BASE

## INSTRUCTIONS

### DESCRIPTION GÉNÉRALE

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### GENERAL DESCRIPTION

Photo Tray or Photo Base are light curing resins in pre-contoured plates for the fabrication of individual impression and functional trays, for bases of bite patterns and bite registration, and for base plates for wax contouring in the total prosthetic technique. The material is ready to use and easy to handle. Photo Tray or Photo Base can be cured under halogen light (about 420-480 nm) or UV light (350-400 nm).

### WARNING !

May cause an allergic skin irritation. Avoid contact of uncured material with skin/mucous membrane and eyes. It may be slightly irritating and in rare cases may cause sensitization to methacrylates.

### METHOD OF USE

#### Fabrication of an individual tray

Mark margins for the individual tray on the plaster model. If necessary, use a space maintainer. If space maintainer is not required, insulate plaster model with a separating liquid. Remove protective foil, mold plate onto the model and cut the desired rims as marked. A handle can be shaped from the excess material.

Depending of the thickness of the tray, polymerize for 4 or 5 minutes in a light curing unit. A longer polymerization time improves the physical properties of the material. The individual tray is ready for use and can be worked upon without delay. A slight inhibition layer, caused by oxygen, can be easily removed with a tissue moistened with alcohol.

#### Fabrication of a base plate

Mark the margin of the planned prosthesis base on the plaster model. Block out undercuts as usual. Insulate plaster model with a separating liquid. Place a plate of Photo Base onto the model, adjust the rims and polymerize the model with base plate for 5 minute in the light curing unit.

Insure sufficient and complete polymerization. A too short exposure time or low intensity of light may lead to a loss of stability.

### STORAGE

To be stored below 25 °C. Avoid direct sunlight. Excess material can be used again if stored under light protection.

**To Order:** DENPLUS Inc.  
333-M Chemin du Tremblay, Boucherville, QC, Canada, J4B 7M1  
Tel.: (450) 641-1330 or toll-free 1-888-344-4424  
Fax: (450) 641-1337  
[www.denplus.com](http://www.denplus.com)

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**Pour commander :** DENPLUS Inc.